

Balboa Comfort LSO Brace

Two Universal Sizing Options - Dual Handle Pulley System

Standard - S thru XL (25" - 45" circumference) Item # LS-627A

Maximum - 2XL thru 4XL (45" - 60" circumference) Item # LS-627B

Code L0627 & L0642 Approved



The *Balboa Comfort LSO Brace* provides superior 5 : 1 compression and stabilization from L-1 to below L-5 vertebra

- Removable rigid posterior panel extends from L-1 to below L-5 vertebra offering equalized compression and support
- Lightweight low profile rigid anterior panel is built into the belt and provides additional abdominal comfort and support
- Flexible pulley system contours and conforms to the patient's body type which allows the patient to adjust easily for desired comfort when standing or sitting
- Easy to use dual handle pulley helps provide equal compression as well as aids patients with arthritic hand issues and muscle weakness



Tips:

- Extend or pull the brace completely out prior to application.
- Initially align dual pulley handle to the side of the brace prior to fitting.
- Adjust tension for sitting or standing.
- Hand wash in warm water, rinse well, towel and air dry completely.

The *NEW Balboa Comfort LSO* is a universal lumbar orthosis which provides sagittal control from an anterior and posterior rigid panel. The *Balboa Comfort LSO* has a flexible pulley system which allows for maximum comfort and fit. A dual handle pulley system allows for the patient to receive equalized compression throughout the brace.

* **Indications include but are not limited to:**

- Low back pain, sprains and strains, and scoliosis
- Lumbar disc displacement, disc degeneration, disc herniation, and compression fractures
- Post-operative laminectomy, post-operative discectomy, IDET procedure, spondylolisthesis, post-operative fusion, spinal stenosis, and osteoporosis



Contoured Padded Posterior Panel provides unsurpassed comfort and support

* Medical advice should be sought on the appropriateness and the condition under which any given spinal bracing system should be utilized.



PROTOSTHETICS
3D PRINTED PROSTHETICS & ORTHOTICS

Protosthetics
617 4th St N, Ste. 1
Fargo, ND 58102
contact@protosthetics.com
(701) 478-2001


Balboa Comfort LSO Brace

Two Universal Sizing Options - Dual Handle Pulley System


Fitting Instructions:

1. Locate Size

Standard
LS-627A

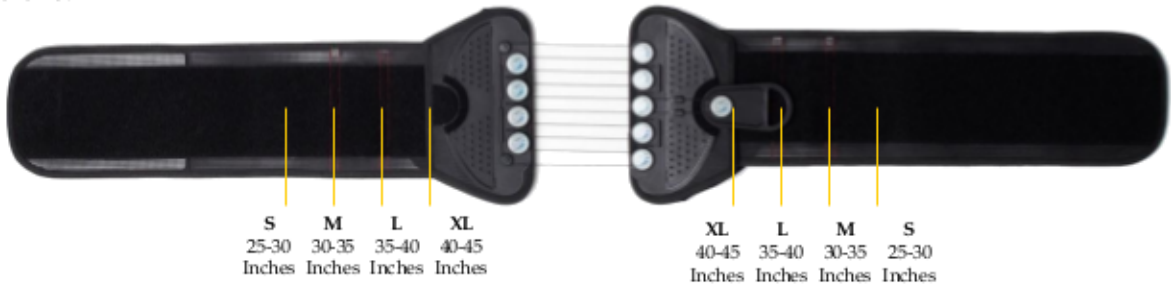

Maximum
LS-627B

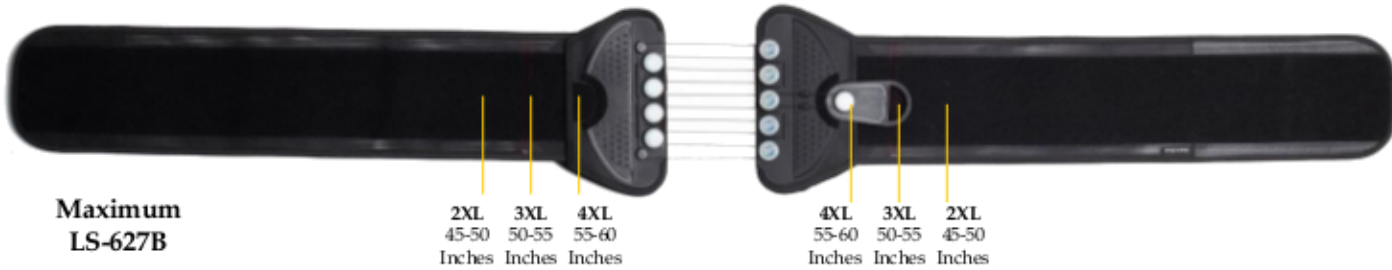

2. Cut to Size



Lift sleeve and remove extension panels to adjust to appropriate size.



Cut to desired length with scissors (prior to cutting, refer to sizing chart for proper length).



If size adjustment is minimal, simply fold the extension panel to desired length.



Once adjustment is made, (cut or folded) simply lift the sleeve and attach the extension to Velcro. Press down to secure.

3. Fit to Size



1. Initially align each pulley handle to the side of the brace prior to fitting. Adjust tension for sitting or standing.



2. Center back panel on the spine with the pointed part of the posterior panel up.



3. Wrap the left side to the front, positioning the front panel. Then wrap right side on top of the left panel to close and attach in front.



4. Pull each side pulley at the same time out and away to desired compression and support.



5. Place the handles on the fabric for a secure fit.



PROTOSTHETICS
3D PRINTED PROSTHETICS & ORTHOTICS

Protosthetics
617 4th St N, Ste. 1
Fargo, ND 58102
contact@protosthetics.com
(701) 478-2001