



ZERO G AFO™ 2.0 OFFLOADING BOOT

Gravity-Defying Innovation

The only prefabricated brace that offers up to 90% offloading of the foot and ankle

The Zero G AFO™ 2.0 ankle foot orthosis (AFO) is a prefabricated, offloading brace that relieves plantar surface pressure from the foot. The Zero G AFO™ 2.0 incorporates a supple, but strong, adjustable, leather calf corset with ratcheting dial adjuster closures. This design provides a total-contact, hydrostatic lift of the inverted cone shape of the calf to offload the foot. The Zero G AFO™ 2.0 brace's patented design, with malleable metal uprights, provides a sturdy, stable substructure that transfers weight from the ground to the calf and not the foot and ankle, providing up to 90 percent non-weight bearing.



Indications

The Zero G AFO™ 2.0 indicated for the following conditions:

- Charcot foot (neuropathic joint disease)
- Arthritis, including Gout / Gouty arthritis
- Osteochondritis Dissecans
- Spina Bifida
- Foot drop and / or ankle instability
- Heel and / or metatarsal ulceration
- Diabetic foot ulcers
- Fractures of the distal tibia, talus, or calcaneus
- Pre or post surgical intervention of the mid and hind foot
- Crushing injuries of the mid and hind foot
- Any condition in which unweighting of the foot or ankle is needed

The Zero G AFO™ 2.0 comes with three components: Calf lacer, silicone interface and the AFO Base.

CALF LACER

- Bi-Valve design with easy Ratcheting Dial adjuster closure system for ease of donning / doffing and provides Infinite adjustment.
- Anterior tongue protection to complete the 360-degree lacer.

Zero G AFO™ 2.0 SILICONE INTERFACE. (Included with calf lacer)

- Parylene® surface coated silicone sleeve provides a soft interface which prevents pressure sores while providing ultimate comfort.

AFO BASE

- Half inch (½") removeable donning pad to ensure a repeatable offloading process.
- Double-action ankle joints for assist / resist or fixed adjustment.
- Footplate pad of 1/8" foam over 1" memory foam foot pad to significantly reduce any pressure applied to plantar surface.
- Adjustable, padded foot cover encases and protects the foot.
- Universal for left or right fitting.
- Suggested L-Codes:

- L1971 – AFO Zero G AFO™ 2.0 Pre-Fab (Base Code)
- L2220 X 2 – Double Action Ankle Joint
- L2265 – Long Tongue Stirrup
- L2397 – Suspension Sleeve Lower Extremity (Interface)



PROTOSTHETICS
3D PRINTED PROSTHETICS & ORTHOTICS



ZERO G AFO™ 2.0 OFFLOADING BOOT

Gravity-Defying Innovation

The only prefabricated brace that offers up to 90% offloading of the foot and ankle

Calf Lacer and Silicone Interface

Product #	Size	Calf Circumference
ZGAL 2.0S	Small	9.5-12.5"
ZGAL 2.0M	Medium	13-15.5"
ZGAL 2.0L	Large	16-19"
ZGAL 2.0XL	X-Large	19-22.5"

AFO Base

Product #	Size	Foot Length
ZGA 2.0S	Small	8.19-9.02"
ZGA 2.0M	Medium	9.17-9.84"
ZGA 2.0L	Large	10.04-10.75"
ZGA 2.0XL	X-Large	10.91-11.75"



How to size for the Calf Lacer (Silicone Interface will be included)

Measure the circumference of the patient's calf to determine proper size on Calf Lacer sizing chart.

AFO BASE:

Measure the patient's foot length to determine proper sizing on AFO Base sizing chart.

NOTE:

The Lacer and AFO Base sizes of the Zero G AFO™ 2.0 can be interchanged for proper fit.



PROTOSTHETICS
3D PRINTED PROSTHETICS & ORTHOTICS